




EvenCare®G2® Blood Glucose System

Performing a Control Solution Test


step
1

Insert test strip to turn on meter.

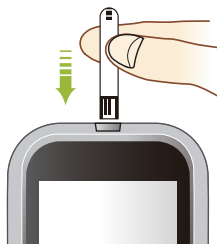

step
4

Bring meter and strip onto the droplet. The meter will beep and start counting down after enough control solution is drawn into the test strip.

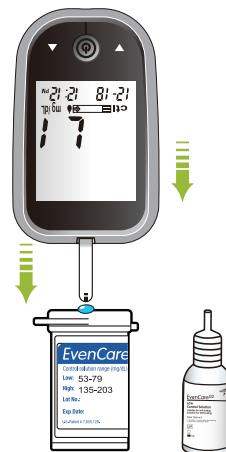


step
2

Press ▼ or ▲ once to enter control solution mode for low or high. "Ctl" icon will display on lower left corner of the LCD, indicating the meter is in control mode.




step
5

Compare result to the low or high range printed on test strip vial.

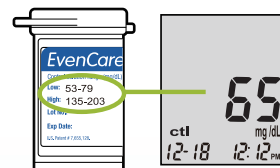

step
3

Invert and gently shake the control solution bottle three times. Discard first drop. Apply second drop to top of test strip vial.

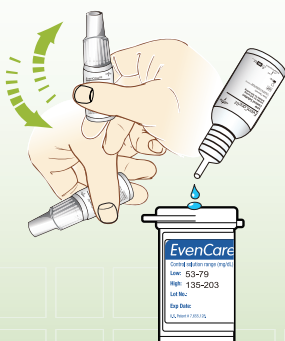

step
6

Remove and discard the used test strip. The meter will automatically turn off. Wipe clean the top of the strip vial.



CAUTION: Failure to follow instruction may cause out of range results.

NOTE:

- Write down the date when the control solution is first opened. Discard three months after the date of the first opening.
- For more information on performing a control solution test, see EvenCare G2 manual, pages 25-28, or call Customer Support at **1-866-274-2131**.